


KIATO Safety Scalpels help to minimize the risk of surgical blade injuries that occur before use, during passing, between steps of the procedure, after use and during disposal. The Safety Scalpel is designed to be used single handedly.

1. Ship/Pass position:



The retractable sheath securely covers the blade fully. The latch must be released to retract the sheath backwards to the Use Position. The sheath does not retract backwards without activation.

2. Use position:

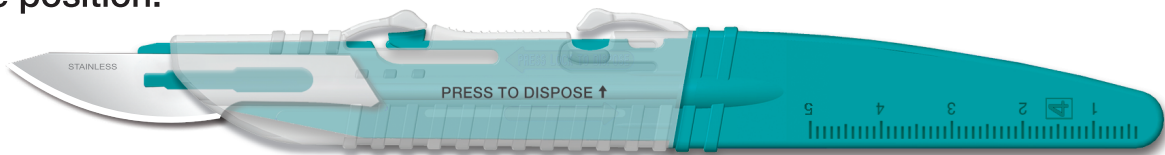

The Surgical Blade is exposed and ready for use. Audible click confirms sheath is fully retracted.




Packed in easy peel rigid blister pouches. 10 scalpels per unit box. Boxes are colour coded for easy identification. 10 boxes per case. 10 case per Master Shipper.

Directions for use


To Use:

Press Release button marked On to depress latch on handle and retract sheath fully until it clicks. The blade is now exposed and the sheath is locked in the "USE" position

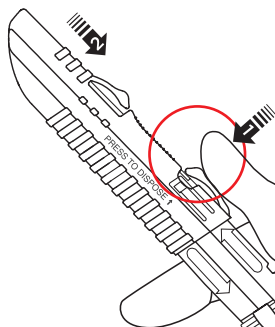

To Pass:


Press Release button marked Off and slide sheath forward to initial position until it locks in the pass position.

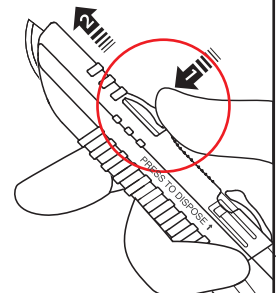

To Dispose


Step 1

Slide Sheath to "pass position". Push dispose button inwards into handle until it clicks & locks.

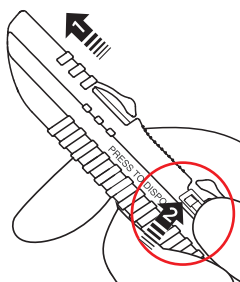


Step 2

Dispose in an approved sharps container.

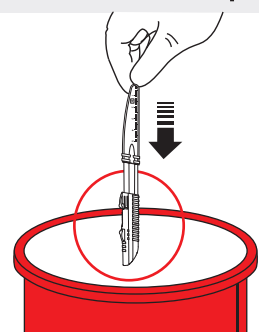

- Premium Swedish Stainless Steel.
- Durable sharpness.
- Clearly printed instructions.
- Cap Safely locked in ship & pass position.

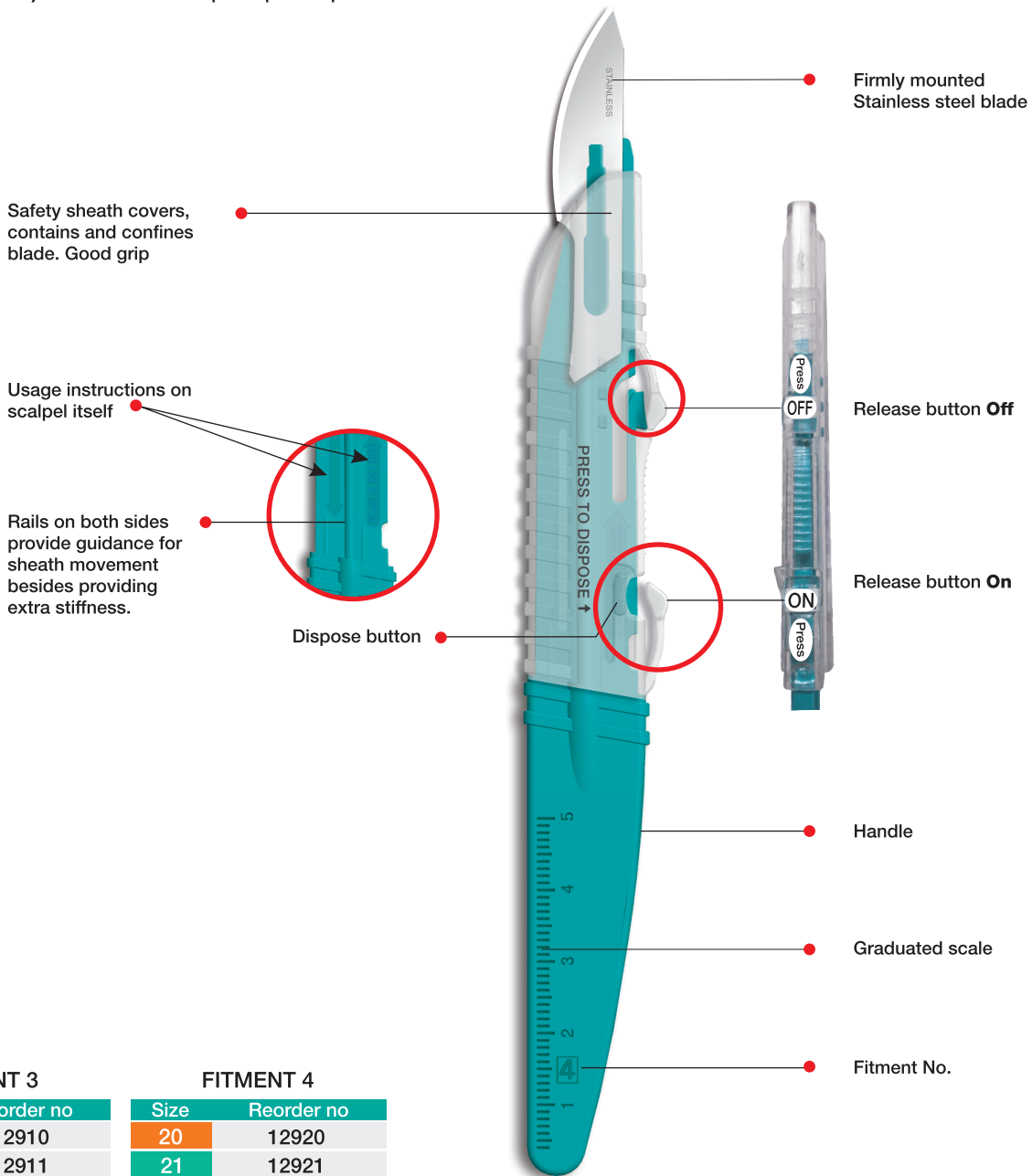

FITMENT 3

Size	Reorder no
10	12910
11	12911
11P	12931
12	12912
15	12915

FITMENT 4

Size	Reorder no
20	12920
21	12921
22	12922
23	12923
24	12924



MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany



0 4 3 4

aditya
DISPOMED



Aditya Dispomed Products Pvt. Ltd.
Plot No. 19 Sector 6 IMT Manesar
Gurgaon Haryana 122050 India
Tel. +91 124 47 64 900
Fax +91 124 47 64 911
info@adityadisposed.com
www.adityadisposed.com

The smart choice for sharp blades

A subsidiary of Kehr Surgical Pvt. Ltd.